



Technical Note myQA TN030 251212 01 / MQ00-000

Subject: myQA config file permissions

Products affected: myQA

Function affected: Security of configuration file in Program Files folder

After the installation of the myQA software, the *IBADos.CSP.Run.Shell.exe.config* file within the Program Files (x86) folder is installed with **Full Control** permissions.

Though this prevents unauthorized users from editing all files in this folder, including the *IBADos.CSP.Run.Shell.exe.config* file, it does not meet the Microsoft security guidelines.

To increase the security level of the myQA application, it is recommended that the IT Administrator change the permission of the *IBADos.CSP.Run.Shell.exe.config* file to "read-only".

TTP: 29096

Please feel free to contact us as follows if you have any further questions:

USA, Canada, Latin America

Phone: +1 786 288 0369

service-usa@iba-group.com

Europe, Middle East, Africa

Phone: +49 9128 607 38

service-emea@iba-group.com

Asia Pacific

Phone: +65 3129 2472

service-apac@iba-group.com