

Technical Note myQA Daily TN003 250829 02 / DQ01-000

Subject: Correct chambers for symmetry and flatness tests

Products affected: myQA Daily

Function affected: False test results due to incorrect chamber information

This Technical Note is to inform our myQA Daily customers of an error in Section 4.1. in the *myQA Daily Instructions for Use* (page 11). In this section, the indicated chambers (including the image) for symmetry and flatness measurements/tests are incorrect.

For symmetry and flatness measurements/tests the following number of chambers are used:

- Field size of 10 cm × 10 cm: 25 chambers.
- Field size of 20 cm × 20 cm: 61 chambers.

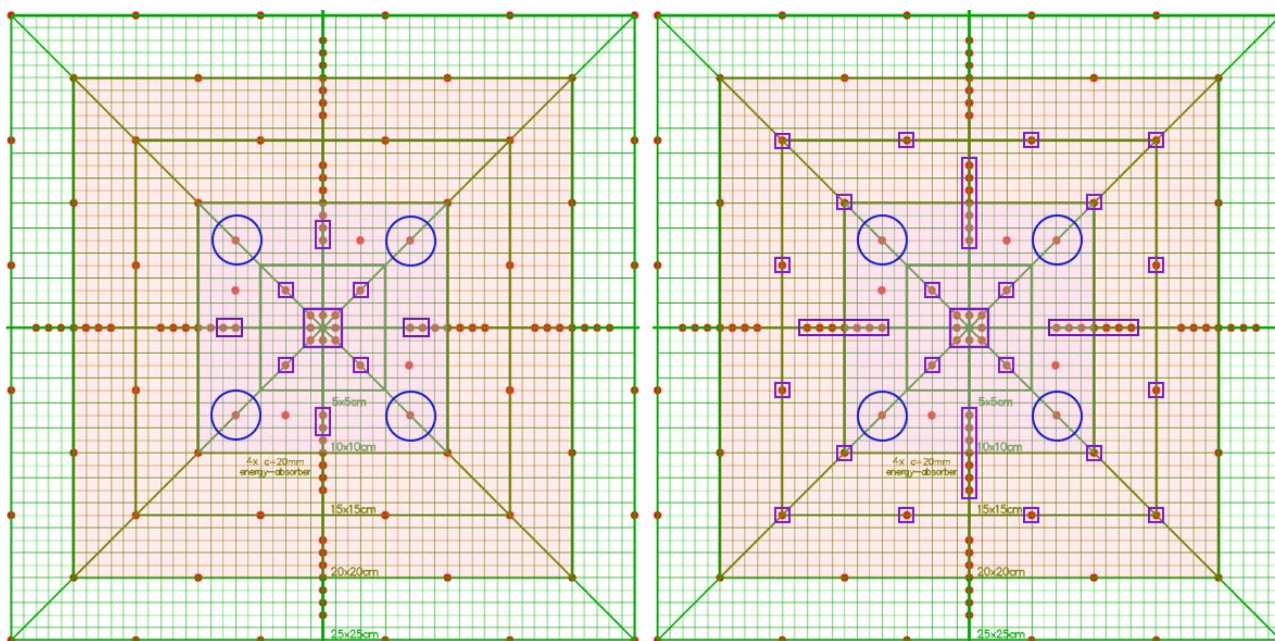


Illustration of the chambers used for symmetry and flatness measurements/tests (highlighted with purple boxes or blue circles); left: for 10 cm × 10 cm field (25 chambers); right: for 20 cm × 20 cm (61 chambers).

Please note, this will be corrected in the next version of the *myQA Daily Instructions for Use*.

TTP: 29070

Please feel free to contact us as follows if you have any further questions:

USA, Canada, Latin America

Phone: +1 786 288 0369
service-usa@iba-group.com

Europe, Middle East, Africa

Phone: +49 9128 607 38
service-emea@iba-group.com

Asia Pacific

Phone: +65 3129 2472
service-apac@iba-group.com