

**Technical Note** myQA Accept TN024 260706 01 / MQ04-001; UQ04-004; UQ04-006; UQ04-007; MQ04-127; MQ04-126; MQ05-000; MQ05-001

**Subject:** Symmetric options for diagonal profiles


**Products affected:** myQA Accept

**Function affected:** Making diagonal profiles symmetric

When using the **Data Handling > Make Symmetric...** function on diagonal profiles, the generated profile may be calculated incorrectly.

To ensure accurate treatment planning data, diagonal profiles created with this function should never be used for any purpose (e.g., RTPS export).

This issue affects software versions 9.2.49 and 9.2.102.

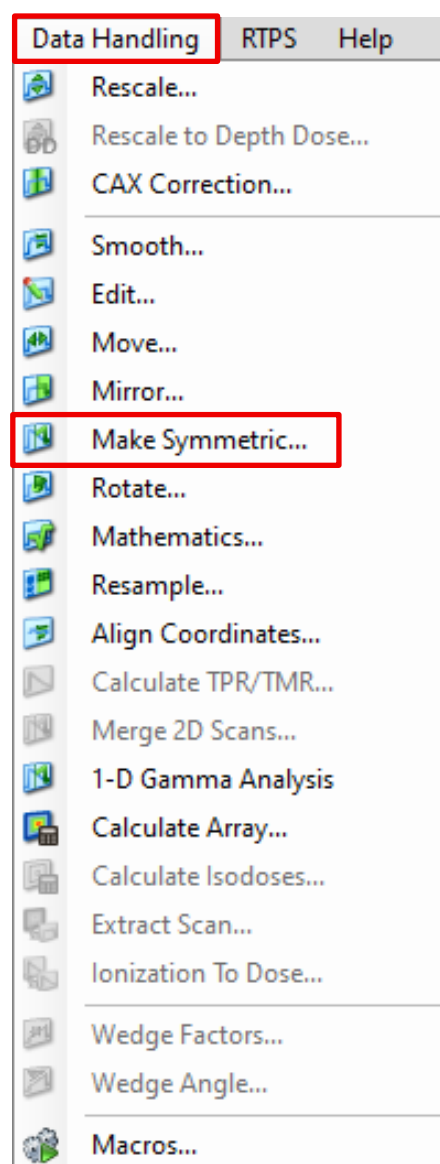

TTP: 30398

**Please feel free to contact us as follows if you have any further questions:**

**USA, Canada, Latin America**

Phone: +1 786 288 0369

[service-usa@iba-group.com](mailto:service-usa@iba-group.com)

**Europe, Middle East, Africa**

Phone: +49 9128 607 38

[service-emea@iba-group.com](mailto:service-emea@iba-group.com)

**Asia Pacific**

Phone: +65 3129 2472

[service-apac@iba-group.com](mailto:service-apac@iba-group.com)